In-situ hardening alloplastic bone graft system

Composition

- easy-graft® CLASSIC contains:
 • Pure beta-tricalcium phosphate (β-TCP) with poly(lactide-co-glycolide) (PLGA)
 • BioLinker (N-Methyl-2-pyrrolidone-solution)

Description

easy-graft® CLASSIC is an alloplastic bone defect filler. It consists of two components: granules (syringe) and BioLinker (ampulla). After mixing, easy-graft® CLASSIC is putty-like and can be applied directly from the syringe into the bone defect. In contact with body fluids, easy-graft® CLASSIC hardens within minutes to form a solid but porous implant.

easy-graft® CLASSIC is biocompatible and osteo-regenerative. For β-TCP and PLGA in adequate form, no inflammatory or immunoreactions are known.

easy-graft® CLASSIC forms a stable and porous body within the defect. The granules already feature a high porosity. After placing the material in the defect, the pore volume allows the uptake of blood. With the integration of blood, important factors for tissue regeneration are incorporated. In a second step, autogenous tissue can grow into the pore structure.

During the initial phase of degradation, easy-graft® CLASSIC may swell by taking up body fluids, thus supporting a close contact to the surrounding bone tissue. When applied in larger defects, this might result in a sensation of pressure by the patient.

easy-graft® CLASSIC contains no animal or human derived substances. β-TCP and PLGA are derived from fully synthetic raw materials.

Indications

easy-graft® CLASSIC should be used for the filling of mostly unloaded and preferably multi-walled dental or maxillo-facial bone defects. The implantation site should be free of infections and soft or granulation tissue. Indications may include:

- Defects after removal of bone cysts
- Periodontal defects
- Augmentation of alveolar crest (e.g. Guided Bone Regeneration, GBR)
- Defects after root end resection
- Extraction defects
- Defects after surgical removal of retained teeth
- Sinus floor elevations
- Defects after removal of autologous bone

Application

Aseptic working methods are essential during application.

easy-graft® CLASSIC is provided in the particle sizes of 500 – 630 µm for smaller defects and 500 – 1000 µm for larger defects.

Dental membranes may be used in combination with easy-graft® CLASSIC at the discretion of the practitioner.

Limitation of use / Precautions**Contraindications**

easy-graft® CLASSIC must not be used during pregnancy.

easy-graft® CLASSIC should not be applied in case of the presence of any diseases or therapies which are detrimental to the healing of bone.

No allergic reactions against easy-graft® CLASSIC or its components are known. However, if an allergic reaction is suspected, the product should not be applied in the affected patient.

Precautions

In order to facilitate the formation of new bone to a maximum extent, easy-graft® CLASSIC should only be implanted in close contact to well vascularized surrounding bony tissue (refreshment of bleeding might be necessary).

Periodontology

Before filling periodontal defects, a sufficient pre-treatment is necessary. This should include sufficient plaque control and a successful local treatment of the periodontal lesion (e.g. root planning, debridement).

Possible adverse effects

Allergic or inflammatory reactions to the implant material or its decomposition products have not been recorded.

Any serious incident that has occurred in relation to easy-graft® CLASSIC should be reported to the manufacturer and the competent authority of the country where the dental professional and/or patient is established.

Interaction with other substances

Not known.

Warnings

easy-graft® CLASSIC is restricted to dental and maxillo-facial applications and to be sold to, or on the order of a licensed physician or dentist.

easy-graft® CLASSIC must not be applied to acute infected defects.

Do not place easy-graft® CLASSIC in direct contact with nerve tissue.

The product has not been clinically tested in children.

Lactation: It is not known whether metabolites of easy-graft® CLASSIC pass into breast milk.

BioLinker contains N-Methyl-2-pyrrolidone. N-Methyl-2-pyrrolidone is irritant to eyes, respiratory system and skin. Wear gloves and protective goggles when working with BioLinker. Change gloves after contact with BioLinker.

After eye contact: In case of contact, immediately flush eyes with copious amounts of water for at least 10 minutes and consult a doctor.

Do not use if package is opened or damaged, or if the expiration date has been exceeded.

easy-graft® CLASSIC cannot be reused. Discard any unused material. Sterility and function are no longer guaranteed in the event of re-use.

Properties

easy-graft® CLASSIC consists of β-TCP, PLGA and BioLinker. Resorption proceeds via three steps: 1) The BioLinker is metabolized and excreted within days mainly through the urine. 2) The absorbable polymer (PLGA) is cleaved by hydrolysis. The degradation products are excreted. 3) The resorption of β-TCP takes place mostly parallel to bone regeneration. Depending on the regeneration potential of the tissue, easy-graft® CLASSIC will be completely resorbed within 5 to 15 months.

easy-graft® CLASSIC supports bone regeneration and bone formation. The application of the material does not guarantee a treatment success. The success of the therapy depends on many factors such as surgical technique and habits: age and the regeneration potential of the patient.

Miscellaneous

easy-graft® CLASSIC is gamma sterilized and intended for single application.

Unused material can be disposed in the domestic waste.

Storage instructions

easy-graft® CLASSIC must be stored in its original packaging. Store between 5 °C to 25 °C. Keep away from sunlight.

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